
## Instructions for use

- EN** more information on [www.fkg.ch](http://www.fkg.ch)
- FR** informations supplémentaires sur [www.fkg.ch](http://www.fkg.ch)
- DE** weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)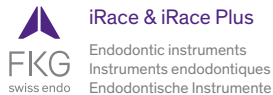


FKG REF. - 99.7AA.10.12A.XX - notice n°100 - 2015/06

## iRace & iRace Plus

Description - Beschreibung

**Taper (1) and ISO size (2) identification**  
**Identification conicité (1) et diamètre ISO (2)**  
**Konizität (1) und ISO Grössen (2) Identifikation**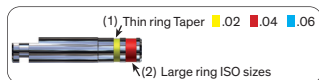

**Depth marks ▲ (in millimeters)**

Available on instruments of 21 / 25 / 31 mm

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**SafetyMemoDisc (SMD)**

Pull off petals to master metal fatigue:

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## EN Golden rules

Speed: 600 rpm - Torque: 1.5 Ncm

- > Long back and forth gentle strokes
- > Light touch, let the instrument work
- > Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal

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## iRace 1 blister includes/contient/enthält: 1 × R1, 1 × R2, 1 × R3

**For most cases**

- straight, slightly curved and/or large -  
*Prepare glide path first, then*

1. Introduce R1 in rotation and reach working length (WL); In case R1 does not reach the WL, do not force and go to step 1 of iRace Plus protocol
2. Continue shaping with R2 up to WL;
3. Finish the shaping with R3 up to WL.

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## iRace Plus 1 blister includes/contient/enthält: 2 × (R1a + R1b)

**For difficult cases**

- severely curved, narrow and/or calcified -  
*Following of iRace protocol step 1*

1. Use R1a to reach WL;
  2. Continue shaping with R1b up to WL;
- When WL is reached, continue from step 2 of iRace protocol.

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iRace Plus

EN

## Sterilization protocol for reusable medical devices

### Non sterile products

#### First use

Carefully follow the rules herebelow  
**Cleaning and Sterilization**

### Sterile products

#### First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

#### Second use:

Carefully follow the rules herebelow  
**Second and following uses.**

### Cleaning

1. Manual cleaning with/without Ultrasonic assistance
- Thorough cleaning allows efficient disinfection & sterilization.
2. Rinse
- Under distilled/demineralized water at least 1 min.
- Then dry the instruments.

3. Inspection - Check
- Sort out damaged or worn out instruments.
4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

### Sterilization

5. According EN/ISO 17664 protocol  
Autoclave: 134°C/273°F,  
2.2 bar during 18 min.

6. Storage
- Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

### Second and following uses

7. Pre-disinfection
- Plunge instruments after use in detergent solution and eventually brush them manually

8. Rinse
- Under running water at least 1 min. Then dry the instruments.
9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on [www.fkg.ch](http://www.fkg.ch)

### General information

#### Pre-/Disinfection DO NOT USE

- Solutions containing:
- phenol (corrosion)
  - aldehyde (blood fixation)
  - di-/triethanolamines (corrosion)

**Sterilization with chemiclav or dry air devices** have not been validated by FKG Dentaire.  
Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

#### Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

A single patient use is recommended to avoid cross-contamination.

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