
- EN** more information on [www.fkg.ch](http://www.fkg.ch)
- FR** informations supplémentaires sur [www.fkg.ch](http://www.fkg.ch)
- DE** weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)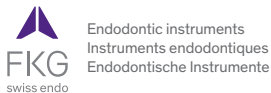

FKG REF. - 99.000.10.02A.XX - notice n°101 - 2015/06

**EN**

## Sterilization protocol for reusable medical devices

General advice:

**Instruments stamped**   
are intended for single use only

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  
0120

## EN Sterilization protocol for reusable medical devices

### Non sterile products

#### First use

Carefully follow the rules herebelow

#### Cleaning and Sterilization

### Cleaning

1. Manual cleaning with/without Ultrasonic assistance

Thorough cleaning allows efficient disinfection & sterilization.

#### 2. Rinse

Under distilled/demineralized water at least 1 min.  
Then dry the instruments.

### Sterilization

5. According EN/ISO 17664 protocol

Autoclave: 134°C/273°F,  
2.2 bar during 18 min.

### Sterile products

#### First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

#### Second use:

Carefully follow the rules herebelow  
**Second and following uses.**

#### 3. Inspection - Check

Sort out damaged or worn out instruments.

#### 4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

#### 6. Storage

Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

### Second and following uses

#### 7. Pre-disinfection

Plunge instruments after use in detergent solution and eventually brush them manually

#### 8. Rinse

Under running water at least 1 min.  
Then dry the instruments.

#### 9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on [www.fkg.ch](http://www.fkg.ch)

### General information

#### Pre-/Disinfection DO NOT USE

Solutions containing:

- phenol (corrosion)
- aldehyde (blood fixation)
- di-/triethanolamines (corrosion)

**Sterilization with chemiclav or dry air devices** have not been validated by FKG Dentaire.

Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

#### Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

 200°C  
(392°F)

A single patient use is recommended to avoid cross-contamination.

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