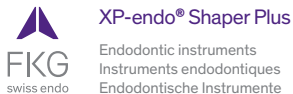
# XP-endo® Shaper Plus sequence

1 blister includes/contient/enthält: 1 x (K-File 10/.02, K-File 15/.02, Shaper 30/.04, Finisher 25/.00)

- EN see www.fkg.ch for additional information
- FR informations supplémentaires sur www.fkg.ch
- DE weitere Informationen auf www.fkg.ch

FKG REF. 99.X00.10.07A.XX - n°109 - 2017/12

## XP-endo® Shaper Plus

Description - Beschreibung

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## EN General information

### Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

### Contraindications :

Titanium nickel instruments must not be used on individuals with a known allergic sensitivity to these metals.

**Adverse reactions :** None known.

### Warnings and precautions :

- In multirrooted teeth, begin with the largest canal.
- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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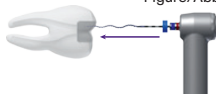
Figure/Abbildung 1


Speed : 800 rpm (800-1000 rpm)  
Torque : 1 Ncm

1. Prior to using the XP-endo® Shaper (XP-S), establish glide path to at least 15/.02. *In calcified/constricted canals or complex curvatures, a glide path to 10/.04 is recommended.*
2. Insert the tip of the XP-S into the canal (Fig. 1), retract slightly and engage the handpiece in rotation mode. *The canal and the pulp chamber should always contain irrigant.*
3. Use gentle strokes to progress down to working length (WL) (Fig. 2); disengage after each stroke. If WL is not reached in 3 to 5 strokes, stop, irrigate, recapitulate and proceed again with XP-endo® Shaper. *Never force the instrument and always keep it spinning and moving while in the canal.*
4. Once WL is reached, irrigate and work the instrument in long in and out gentle movements to WL for another 10 strokes (Fig. 3). The final apical dimension is now at least 30/.04.

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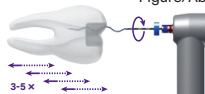
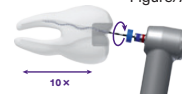
- Irrigate the canal in order to eliminate suspended debris.
- Confirm the final apical dimension with a 30/.04 gutta percha point.
- If a larger preparation is required, use an appropriate Race instrument to obtain the desired final preparation.
- Apply your regular irrigation protocol.

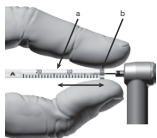Once the tooth has been fully shaped please use the XP-endo® Finisher (XP-F).

Figure/Abbildung 4

- In multirrooted teeth, begin with the largest canal. The canal should always contain irrigant. However, avoid filling the access cavity with irrigant before insertion of the file inside the canal. This is to prevent premature expansion of the file.
- Fix the canal working length (WL) by using the plastic tube (a) to adjust the end stop (b) (Fig. 4).
- Cool the XP-F down inside the tube using a cold spray.
- Put the XP-F in rotation mode and remove it from the tube by applying a lateral movement to ensure the XP-F remains straight (Fig. 5). Turn off the rotation.
- The surface of the tube may be touched with the operators fingers only at its end, on FKG logo. This is to avoid warming the file.*
- If the file is straightened outside the tube use an alcohol soaked gauze to avoid warming of the file.*
- Insert the XP-F tip into the first canal of the tooth (Fig. 6). Once it is inside, turn on the rotation and slowly insert the file into the canal (Fig. 7). Add irrigant to the access cavity.
- In case of any difficulty inserting the file inside the canal of multirrooted teeth, make sure to direct the tip of the expanded file towards the cusp related to the canal (e.g. MB cusp for the MB canals).*
- Use the XP-F for approximately 1 minute, using slow and gentle 7-8 mm longitudinal movements to contact the tip of the file on the full length of the canal (Fig. 8). Make parietal movements during the procedure. Avoid removing the file from the canal during this procedure.
- After 1 minute, remove the XP-F from the canal while it is still in rotation.
- Irrigate the canal to remove the suspended debris. A syringe tip 31G is recommended (or no bigger diameter than 30G).

To continue treatment inside a narrower canal of the same tooth previously shaped :

- Clean the XP-F and place it back inside the measurement tube.
- Begin again the procedure at step 10.

When the full cleaning of a tooth is completed, proceed to the next phase of treatment :

- Obturate with gutta percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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