

Finisher

Cleaning Power Tornado

Experience a powerful clean that stands out. This unique mechanical cleaning device effectively reduces bacteria up to 98%*. Impressive, isn't it? Finisher is able to remove filling material too. A universal solution so revolutionary that you'll become a cleaning fanatic. That's how we empower endo.

Change the Game

* Azim, Adham A et al. "Efficacy of 4 Irrigation Protocols in Killing Bacteria Colonized in Dentinal Tubules Examined by a Novel Confocal Laser/Scanning Microscope Analysis." Journal of endodontics vol. 42.6 (2016): 928-34. doi:10.1016/j.joen.2016.05.009

Discover why:
fkg.ch/power



Finisher




Finisher
25/00


800-1000RPM – 1Ncm

FKG
swiss endo


A universal solution

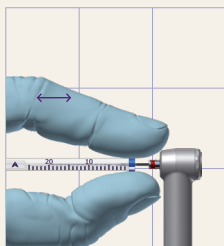

- Shaping min. \varnothing 25.
- Set the working length by using the plastic tube.

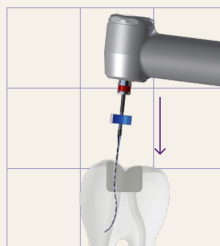

- Start with the largest canal.
- Insert the instrument pointing towards the cusp and start the motor.

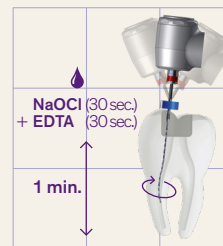

- Clean the entire canal for 1 minute with long and gentle strokes (7-8 mm).
- Proceed to final rinse.

Cleaning Achieved



More info



STERILE R

