

Rotary

# Sharp Control

A 1-2-3 sequence  
or a tailored one?  
Stay sharp,  
keep control.

**RACE® EVO**

# Innovation in the Heart of the Swiss microtech valley

Since 1931, in the heart of the Swiss microtech valley, we have been pushing the limits of endodontics by combining technology and innovation.

Our mission is to provide solutions that respect the complex anatomy of teeth while improving the treatment efficiency. With game-changers such as XP-endo Rise, RACE EVO, R-Motion, Finisher and TotalFill, we have transformed how endodontics is practiced, giving clinicians better tools to meet challenges with confidence.


Didier  
Devaud  
General  
Manager




## Rotary

# Sharp Control

RACE EVO is so precise, everything feels effortless. A short 1-2-3 sequence and wide file choice enables you to handle most situations, from the routine, to the challenging complex cases with confidence. RACE EVO gives you absolute control and allows you to tailor your procedure at any time. You'll wish everything could be so easy to handle. That's how we empower endo.

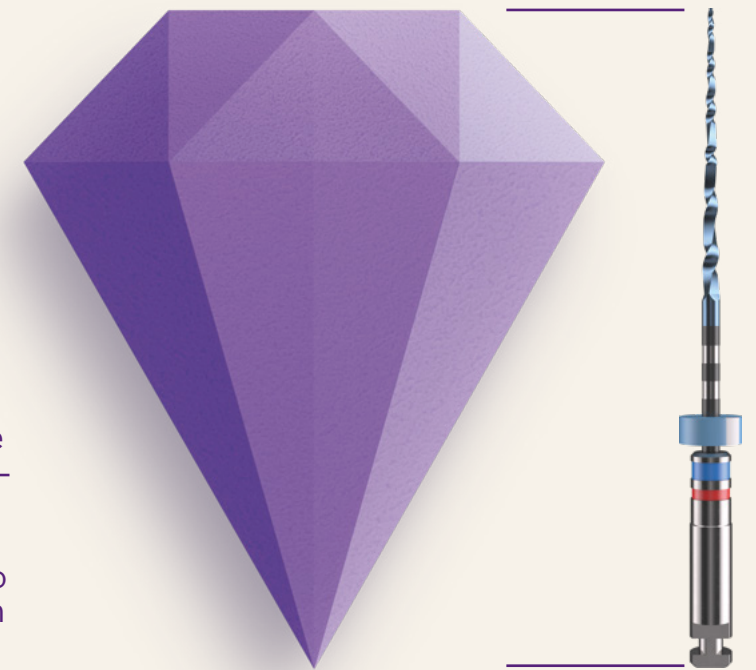


We offer confident precision to make everything feel effortless. A straightforward 1-2-3 sequence and wide file choice enables you to handle most situations, from the routine, to the challenging complex cases with confidence.

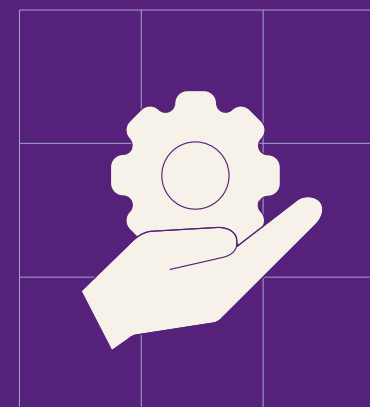


### Absolute control

Made with precision in mind, RACE EVO's unique design ensures an effortless user experience, so you stay sharp and in control.

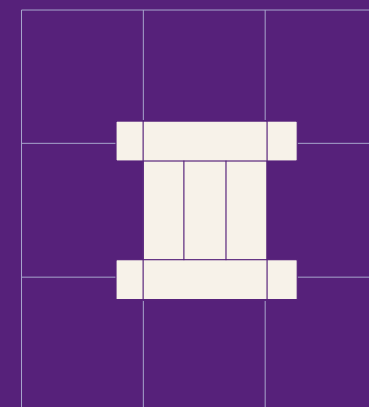

### Simple 1-2-3 sequence

RACE EVO simplifies the vast majority of root canal treatments by offering a straightforward 3-file sequence and uniform motor settings across the range.

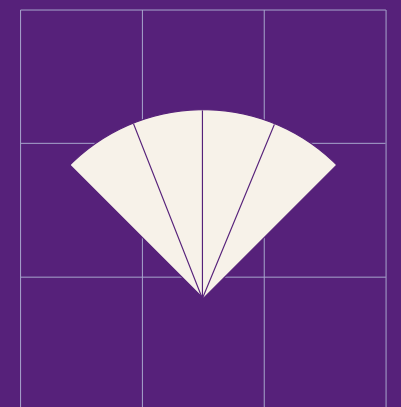

### Wide range

RACE EVO is a comprehensive solution for most clinical scenarios. Its wide and diverse range empowers you to create your own custom sequence(s).





# Absolute control

With their unique and user-friendly RaCe profile, RACE EVO files offer exceptional, effortless maneuverability, giving you full control during procedures.

Building on over 20 years of proven success, RACE EVO's alternating cutting edges reduce the screw-in effect and increase safety during continuous rotation.

With enhanced control, you are always the master of the file — minimizing the risk of iatrogenic errors. RACE EVO is a tangibly different experience, with a smooth, controlled progression through the root canal that sets a new standard in the tactile experience of endodontics.



# Wide range

RACE EVO offers a comprehensive solution for most clinical scenarios, giving you the flexibility to tailor treatment to the individual needs of your patients. With a wide and varied range, you can create a customized sequence to suit the specific cases you encounter.

The RACE EVO range consists of 18 references, including opener, glide path, and shaping files, with an additional four non - heat - treated (RaCe) glide path files.

This unique glide path collection provides various flexibility and rigidity options, helping you choose the perfect file for even the toughest cases.

# Simple 1-2-3 sequence

RACE EVO offers an impressively simple solution: a 3-file sequence that handles most cases.

With just one glide path file followed by two shaping files, your endodontic workflow is straightforward, and requires few instruments.

A straightforward protocol and uniform motor settings across the whole range means less complexity for you and more comfort for your patients. RACE EVO is the height of simplicity, designed to make your job easier.



# The importance of speed

"The RACE design was developed by FKG, so these reamers with alternating cutting edges are the perfect ally for high-speed performance in continuous rotation. This makes RACE EVO files incredibly controllable, allowing users to minimize the risk of error.

Contrary to popular belief, high speed isn't the enemy. Consider a lumberjack cutting a tree: if he uses his chainsaw at a low speed, he'll feel significant resistance and struggle to achieve his goal.

Conversely, if he cuts the same tree with a chainsaw operating at full speed, he'll do it effortlessly and fast, in absolute control. To try RACE EVO is to adopt it!"

Dr. Kleber Carvalho  
DDS, MSc, PhD / Sao Bernardo  
do Campo Brazil






Curious?  
Here's more details  
about RACE EVO:

One  
Sequence for  
Most of your  
Cases



GLIDE  
PATH


RE1



15/.04

INITIAL  
SHAPING



RE2



25/.04

FINAL  
SHAPING

RE3 4%      RE3 6%











 or 

30/.04      25/.06

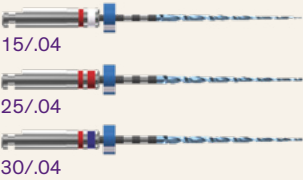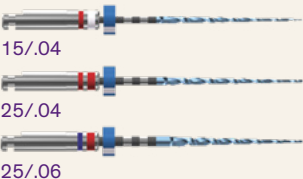




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



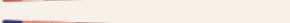

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

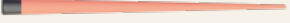



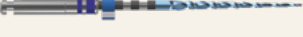





Select the final shaping file based  
on your therapeutic approach.

# References

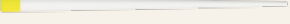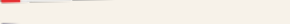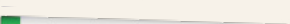

SEQUENCES			
RACE EVO 4% sequence (3 files per blister)			
			
15/.04	}	21mm	S1.7B0.00.SCV.FK
25/.04		25mm	S1.7B0.00.SCW.FK
30/.04		31mm	S1.7B0.00.SCX.FK
RACE EVO 6% sequence (3 files per blister)			
			
15/.04	}	21mm	S1.7B0.00.SCY.FK
25/.04		25mm	S1.7B0.00.SCZ.FK
25/.06		31mm	S1.7B0.00.SDA.FK
REFILL			
RACE EVO 15/.04			
	6×	21mm	S1.7B0.00.0DG.FK
	6×	25mm	S1.7B0.00.0FZ.FK
	6×	31mm	S1.7B0.00.0DH.FK
RACE EVO 25/.04			
	6×	21mm	S1.7B0.00.0DI.FK
	6×	25mm	S1.7B0.00.0GB.FK
	6×	31mm	S1.7B0.00.0DJ.FK
RACE EVO 30/.04			
	6×	21mm	S1.7B0.00.0DK.FK
	6×	25mm	S1.7B0.00.0GC.FK
	6×	31mm	S1.7B0.00.0DL.FK
RACE EVO 40/.04			
	6×	21mm	S1.7B0.00.0DM.FK
	6×	25mm	S1.7B0.00.0GE.FK
	6×	31mm	S1.7B0.00.0DN.FK
RACE EVO 50/.04			
	6×	21mm	S1.7B0.00.0DO.FK
	6×	25mm	S1.7B0.00.0GF.FK
	6×	31mm	S1.7B0.00.0DP.FK
RACE EVO 25/.06			
	6×	21mm	S1.7B0.00.0DQ.FK
	6×	25mm	S1.7B0.00.0GH.FK
	6×	31mm	S1.7B0.00.0DU.FK
RACE EVO 30/.06			
	6×	21mm	S1.7B0.00.0DV.FK
	6×	25mm	S1.7B0.00.0GI.FK
	6×	31mm	S1.7B0.00.0DW.FK
RACE EVO 35/.06			
	6×	21mm	S1.7B0.00.0DX.FK
	6×	25mm	S1.7B0.00.0GJ.FK
	6×	31mm	S1.7B0.00.0GU.FK

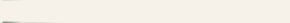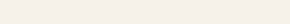● 4% approach      ● 6% approach

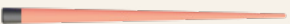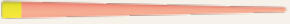PAPER POINTS		
Box of 200 Paper Points		
	30/.04	08.610.04.030.BC
	40/.04	08.610.04.040.BC
	50/.04	08.610.04.050.BC
	25/.06	08.610.06.025.BC
	30/.06	08.610.06.030.BC
	35/.06	08.610.06.035.BC
GUTTA PERCHA		
Box of 60 Gutta Percha		
	30/.04	08.600.04.030.RE
	40/.04	08.600.04.040.RE
	50/.04	08.600.04.050.RE
	25/.06	08.600.06.025.RE
	30/.06	08.600.06.030.RE
	35/.06	08.600.06.035.RE
TOTALFILL® BC POINTS™		
Box of 60 BC Points		
	30/.04	08.650.04.030.RE
	40/.04	08.650.04.040.RE
	50/.04	08.650.04.050.RE
	25/.06	08.650.06.025.RE
	30/.06	08.650.06.030.RE
	35/.06	08.650.06.035.RE
TOTALFILL® BC SEALER™		
Syringe 2g + 15 TotalFill BC Tips		
		08.650.00.007.XX
TOTALFILL® BC POINTS™ HIFLOW		
Box of 60 BC Points HiFlow		
	30/.04	08.650.04.030.HF
	40/.04	08.650.04.040.HF
	50/.04	08.650.04.050.HF
TOTALFILL® BC SEALER™ HIFLOW		
Syringe 1.5g + 15 TotalFill BC Tips		
		08.650.00.008.XX

Find a partner:



# Empower Endo



FKG REF 987AA0011AEN\_REV2\_2025-05-23 - © FKG Dentaire Sàrl - Pictures of our products are not contractual

## Welcome to the heart of the Swiss microtech valley.

For nearly a century, we have been constantly re-inventing and challenging what we thought we knew.

We strive to create innovations for new ways of working: more respectful, more efficient, more agile, more ergonomic, and safer—serving new generations, promoting new visions of life, for us, for you, and for patients.

We embrace change that empowers endo.

FKG | Change the game.

Enter our game-  
changing universe



Check out  
our website



[www.fkg.ch](http://www.fkg.ch)

FKG Dentaire Sàrl  
Le Crêt-du-Loche 4  
2322 Le Crêt-du-Loche  
Switzerland

+41 32 924 22 44  
[info@fkg.ch](mailto:info@fkg.ch)