



Irrigation

Flow Surfer

Endodontic is about precision and flow - and nothing should disrupt your rhythm. With every rinse, every switch, and every pause, momentum is lost. Dual Rinse HEDP changes that. With a single step, it cleans, dissolves, and prepares the canal, without breaking the flow. Fewer syringes. Fewer transitions. Just pure, uninterrupted motion from start to finish. It's time your irrigation moved with your pace. One capsule. One protocol. One less thing to think about. That's how we empower endo.

Discover why:
fkg.ch/flow




Dual Rinse® HEDP




FKG
swiss endo




- Place 10 ml of sodium hypochlorite (NaOCl) into a sterilized mixing cup.

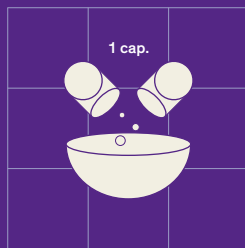

- Open a capsule of **Dual Rinse HEDP** and place the powder into the mixing cup.

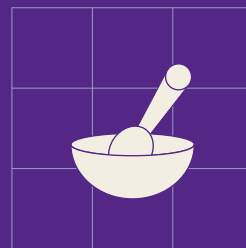

- Stir the NaOCl/**Dual Rinse HEDP** with a sterile instrument until it has completely dissolved.

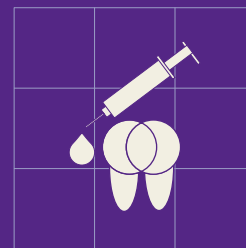

- Use the solution in accordance with the endodontic rinsing protocol.



More info

