



Rooter® X4000 – Frequently Asked Questions

How many free memories can I set parameters on my Router® X4000?

- *You can customize the library and set as many sequences or file parameters as you like. The only limitation is the device's available storage capacity.*

Can I connect an external apex locator to my Router® X4000?

- *The Router® X4000 comes with a fully integrated apex locator. No additional apex locator can be connected to the main unit or handpiece.*

Can I adjust the volume of the beeper?

- *Yes, the sound of both the main unit and the handpiece can be adjusted to three different intensity levels, as well as a silent mode.*

Does the Router® X4000 can be used with other reciprocating files systems than R-Motion?

- *The device is fully programmable and can be set to a variety of reciprocating angles and speeds, thus meeting all current reciprocating motion requirements.*

What is the weight of the handpiece with contra-angle?

- *The weight of the handpiece and contra-angle is 125 grams.*

What is the technology of the mini-motor integrated in the Router® X4000?

- *It is a brushless motor.*

Does the Router® X4000 have an “Automatic Power Off” function?

- *Yes, the Router® X4000 automatically powers off after the set timer, which you can adjust from 30 to 90 minutes. In addition, the device features a ‘Screen Sleep’ mode that automatically turns off the screen after 5, 15, or 30 minutes, helping to save energy.*

When fully charged, how long can I use the unit?

- *Time may vary depending on the usage. The battery capacity is 2600 mAh for the main unit and 1200 mAh for the handpiece.*

How often should I calibrate the device?

- *It is recommended to calibrate the torque on the contra-angle when replacing it or after every 1-2 months of use.*

Does the Router® X4000 has an “Apical Auto Reverse” function?

- *Yes, the motor automatically reverses the rotation direction when the file reaches the reference point set by the user. The motor goes back automatically to initial rotation direction when the file is pulled away from the reference point.*

What are the features of the original contra-angle?

- *It is a 4,7:1 contra-angle offering 360° rotation.*

Can I use another contra-angle than the original one?

- *FKG Dentaire SA does not recommend to use of other contra-angle with the Router® X4000 endo motor than genuine ones.*

How do I clean, disinfect and sterilize the Router® X4000 components?

- *Please consult the Instruction For Use for any questions relating to cleaning, disinfection and sterilization of the Router® X4000 components.*



Empower Endo